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**Important:** Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to UH 1100 „Processing manual implants and instruments“. For individual instruments which are correspondingly marked () , we refer to the „Assembly and disassembly instructions with special cleaning instructions“. These can be downloaded any time free of charge at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)

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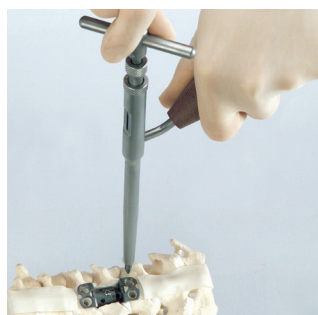
### osmium™ screw

The osmium™ screw (CS 1303-12, -14, -16, -18) is a two-component system consisting of a screw and a bolt. The implant is available in four different lengths (12, 14, 16, 18 mm) with an outer diameter of 4 mm. The length is easily recognizable according to colour: 12 mm grey, 14 mm golden, 16 mm green and 18 mm blue. Expansion of the screw is achieved by inserting the bolt. Hyper-expansion of the screw is not possible.

### Drill hole for osmium™ screw

The drill hole is prepared using the drill, Ø 2.5 mm (CS 1323) through the depth stop (CS 1324, consisting of CS 1324-1 / -2 / -3). Drill under C-arm control, if necessary.

### Tapping for osmium™ screw

The tap for the screw is prepared using the tap for screws, Ø 4 mm (CS 1326) through the depth stop.

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**Important:** Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to UH 1100 „Processing manual implants and instruments“. For individual instruments which are correspondingly marked (△), we refer to the „Assembly and disassembly instructions with special cleaning instructions“. These can be downloaded any time free of charge at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)

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**Instrumentation with osmium™ screws**

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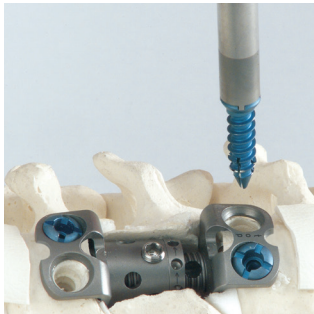
First remove the bolt of the osmium™ screw with the screwdriver (CS 1329). The insert (CS 1310-3) is placed in the screwdriver (CS 1310-1) and the osmium screw is locked. The prongs of the screwdriver must be fitted to the grooves of the screws.

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
Using the screwdriver (CS 1310-1) or the tightener (CS 1322) the screws can be further tightened before being expanded, in order to achieve an even support of the plates.

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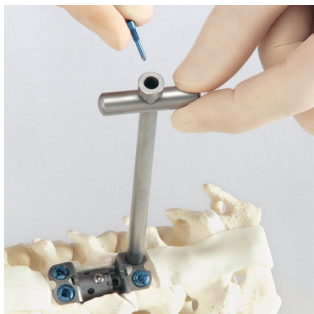The screwdriver for osmium™ screw (CS 1310-1) without the insert (CS 1310-3) is placed onto the screw and the bolt is inserted into the shaft of the instrument.

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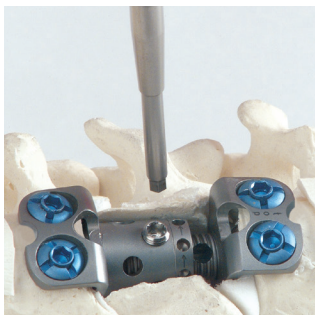
### Fixing the osmium™ screws

The bolt is locked using the screwdriver, hex 2.0 mm (CS 1329) and the osmium screw is expanded.

**Important:** The instrument must remain engaged on the grooves of the osmium screw, in order to prevent that the screw corotates.

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### Locking the implant ADDplus™

After checking the correct positioning and height of the ADDplus™ the locking screw is tightened by using the screwdriver (CS 2261).

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