## Supplement für OP-Technik, perkutaner Zugang, WS 3801-02 Supplement for Surgical Technique, percutaneous approach, WS 3801-02

**System: uCentum™ comprehensive posterior system**

E02971-02

CS 6601-07

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#### Cement augmentation

In the uCentum™ system all cannulated screws (except for screws with Ø 4.5 mm) are provided with perforation for cement augmentation.

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#### Use of the adapter for injection for bone cement applicator

The adapter for injection, percutaneous, for bone cement applicator (E02971-02) can be used alternatively for adapter for injection, percutaneous (CS 3850-02).

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#### Mounting the adapter

Cement augmentation via the uCentum™ screw requires that a guide wire be used.

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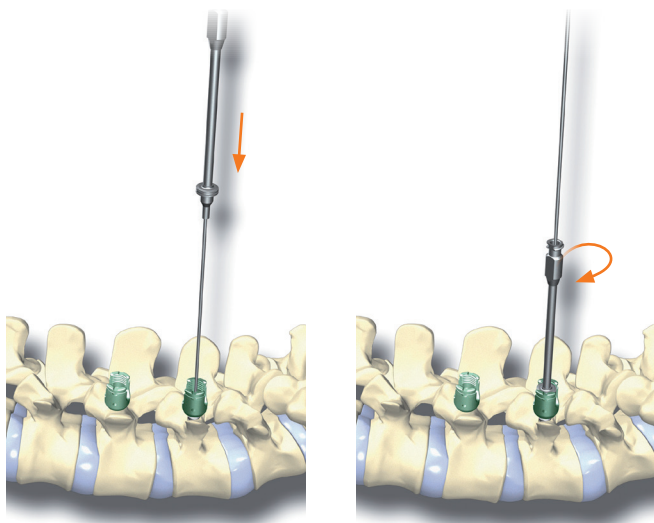
The user has to ensure, that the latest versions of the instructions for use, surgical techniques and corresponding supplements are on hand and considered. For cleaning, disinfection and sterilization of non-sterile implants and instruments, the „Processing manual implants and instruments“ UH 1100 is to be considered. For individual instruments which are correspondingly marked , the „Assembly and disassembly instructions with special cleaning instructions“ have to be considered. These are available at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)

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The guide wire is used to facilitate assembly and ensures the patency of the screw cannulation. The adapter (E02971-02) is placed on the screw via the guide wire and secured in the screw head. Then the guide wire can be removed. All adapters have to be secured to the screws before starting the augmentation.

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### Applying the cement

The cement is applied through the bone cement applicator (CS 6601-07). First the bone cement applicator (CS 6601-07) is filled with cement and inserted into the adapter for injection for bone cement applicator (E02971-02).

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With the aid of the tamper the cement is then injected through the screw base into the vertebral body. To prevent cement back flow, leave the adapters on the screws until the cement.

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**!** **Important:** Lateral C-arm control is recommended.

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**⊗** Adapter and bone cement applicator are for single use only.

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**i** The user has to ensure, that the latest versions of the instructions for use, surgical techniques and corresponding supplements are on hand and considered. For cleaning, disinfection and sterilization of non-sterile implants and instruments, the „Processing manual implants and instruments“ UH 1100 is to be considered. For individual instruments which are correspondingly marked **i**, the „Assembly and disassembly instructions with special cleaning instructions“ have to be considered. These are available at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)